



Your
Time is
Precious.

eyecon®

...e it
...unt.





Enhancements 2023



NEW

2.5x ENHANCED PERFORMANCE

Intel i3-1115G4E motherboard on all units



DOUBLING TRANSACTION STORAGE

Larger 256GB hard drive

- **9420** - 2,000,000 transactions with B&W photos
- **9430** - 140,000 transactions with B&W and color photos



FASTER UPDATES AND BACKUPS

3x USB 3.0 ports



MICROSOFT WINDOWS 10 IOT 2021 LTSC

Microsoft service updates through 2031



SQ EXPRESS 2022

Database for transaction storage



9420




9430







A Look Into The Future Of Pill Counting

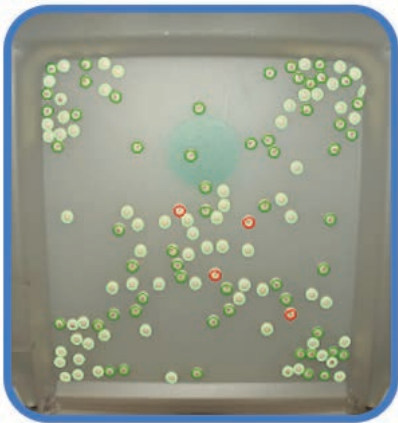What is Optical Pill Verification (OPV)?





With the Optical Pill Verification software running on the Eyecon 9430, it will analyze the colored photos during the fill process to provide error detection based on size, shape, pill color and markings. It will alert the pharmacist or technician of a potential drug mismatch.



Pharmacist key benefits:

-  **FRONT COUNTER** - Patient clinical consulting focus
-  **MULTI-SITE** - Pharmacy level loading
-  **TELEPHARMACY** - Off-site for rural locations
-  **CENTRAL FILL** - Remote pharmacist verification

OPV Results:

-  **Red** - Unvalidated or color not within tolerance
-  **Red** - Color validated, but incorrect markings detected
-  **Light Green** - Color validated, but markings are not detected
-  **Dark Green** - Color validated, and correct markings are detected


Allow for your pharmacy to be front and center for patient consulting and level loading.





Automate In Color. It Counts.

Patients Deserve Both Quality Care
& Optimal Outcomes

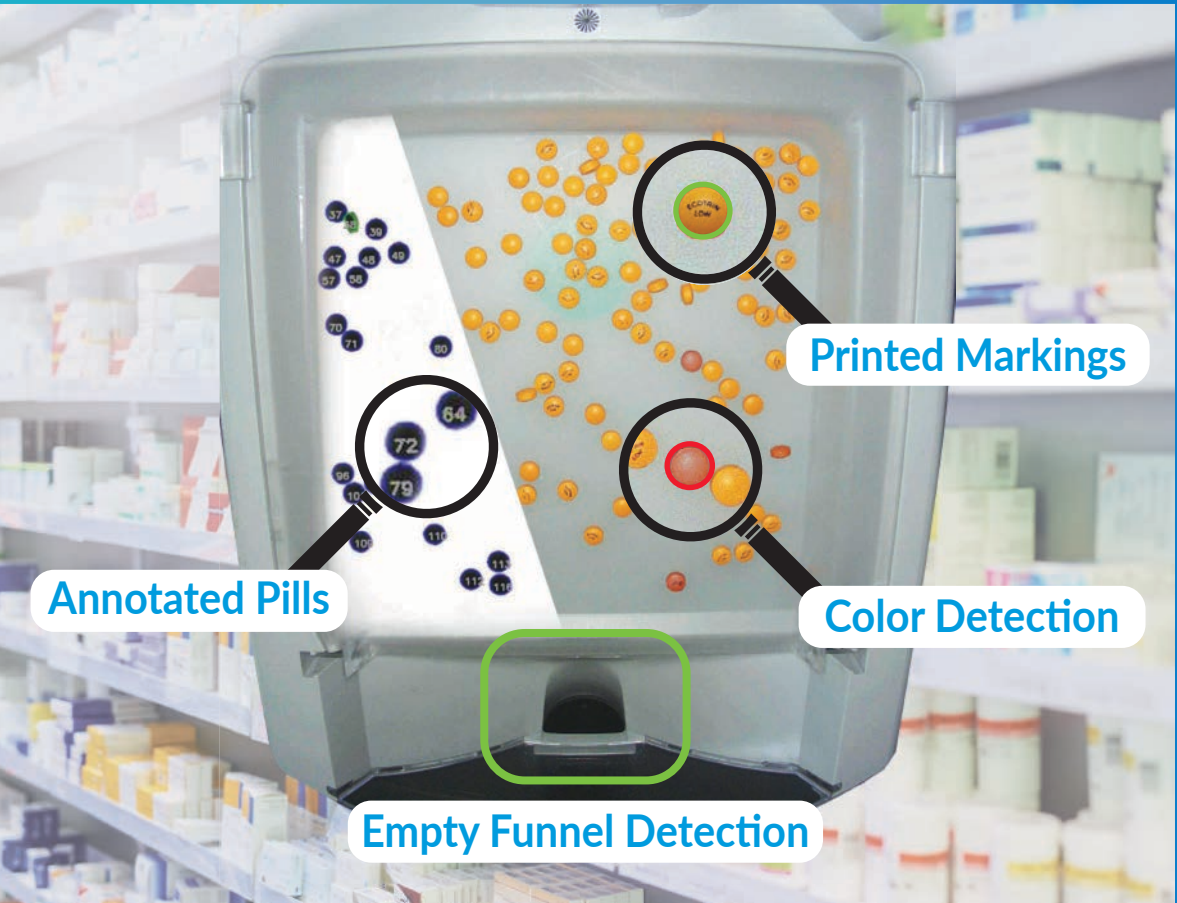

AUTOMATE IN COLOR. IT COUNTS.

**No matter your challenge,
Eyecon is your solution.**



Automate in color. Allow for
your pharmacy to be front and center
for patient consulting and level loading.



Automate in Color

Premium color images allow for transparency
and safety throughout the filling process.



Optical Pill Verification

Enhancing patient safety with advanced features.
Detecting potential drug mismatch via size,
shape, color, and markings analysis.



Streamline your Connection

Integrate your PMS for safety and security. Over
30 PMS interfaces supported with Eyecon.




RX AND PILL COUNT VALIDATION

SAFETY – Improves recognition and identification of incorrect pills, foreign matter and tablet fragments.

EASY TO USE – Quick, easy, and exact. Ready for work 24/7.

VERSATILITY – Counts all sizes and shapes of capsules and tablets. The Eyecon supplies validation and inventory of ointments, creams, solutions, and pre-packs.

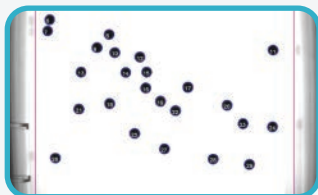


MAXIMUM ACCURACY

IMAGES – Every Rx transaction has an annotated pill tray photograph with individually numbered pills.

INTEGRATION – The Eyecon connects to your Pharmacy Management Software (PMS).

REPORTING – You can view all Rx transactions and associated photos via the Eyecon interface and export when needed.

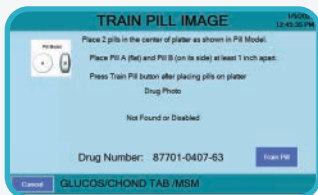

EASY TO USE

UPDATES – Automatic quarterly updates of the inbuilt drug database and software included.

LEARNING – Train the Eyecon to count custom or undocumented pills.

CLEANING – Quick and easy to clean. Simply wipe and go.



What does Eyecon® bring to the table?



SPEED

Fastest Pill Counter Available



ACCURACY

99.99% Accurate 24/7



PEACE OF MIND 24/7

- ✓ Correct Rx
- ✓ Correct Drug & Strength
- ✓ Exact Count



EYECON BENEFITS

eyecon®



Safety brings peace of mind.

eyecon



SAFETY:

The Eyecon features automatic, visual warnings which notify the operator when specific situations occur.



WRONG FILL QUANTITY:

When completing an Rx in validation mode, if the displayed quantity does not match the prescription quantity, a display warning allows the operator to correct the fill amount. This prevents a common cause of fill errors when the operator counts the incorrect quantity.



DOUBLE-COUNT AND BACK-COUNT:

The Eyecon has the option to either double count your Rx fills and/or back-count the balance in your stock bottle after filling an Rx. Adjustments to the Eyecon can be based on the DEA classification level as compared to the DEA level specified in the Eyecon set up for each feature.



PARTIAL FILLS:

Eyecon tracks partially filled prescriptions so when either the correct or generic substitute inventory becomes available; the balance of the partially filled Rx can be filled. A display warning of the previous partial fill allows the operator to dispense the correct balance of the Rx. The Eyecon records every transaction including photos.



GENERIC SUBSTITUTIONS:

Allow the operator to dispense a generic equivalent to the branded Drug Number sent to the Eyecon. A display warning allows the operator to accept or refuse the substitution. You also have the choice





Eyecon networked for success

CONNECTIVITY

The Eyecon has a built in Ethernet interface allowing connectivity on existing networks and beyond. This functionality means that not only can you connect to a range of Pharmacy Management Software (PMS) systems, but also allows you to use the Eyecon as part of your telepharmacy offering.

PMS INTEGRATION

Integration with the PMS means having a smoother workflow process while improving safety and security. Automating the data handling between the PMS and Eyecon means digitally recording every step of the Rx process including each verification process. Verification happens at every stage to ensure the correct Rx, Drug number, Stock bottle and quantity.

INVENTORY UPDATES

The Eyecon can also transfer a summary of all physical inventory counts to update perpetual counts quickly and accurately in an inventory management system.

TELEPHARMACY

Eyecon seamlessly integrates with your existing telepharmacy solution by making it easy for remote verification of the Rx by recording HD images of the annotated tray image, color image of the pill tray, color image of the filled vial and a color image of the vial label. All images are associated with the Rx which include; date, time and dispensing user information.



eyecon®



PHYSICAL INVENTORY

Reduce the time to complete a physical inventory and cycle count by over 70%. Fast and exact counting along with digital recording.

- Eliminates excess inventory.
- Minimizes physical inventory time by over 70%.
- Reduces inventory variances.
- Decreases physical inventories due to increased prescription filling accuracy.
- Improves employee morale by automating pill counting and allowing for other focuses within facility.
- Eyecon also counts ointments, creams, solutions and pre-packs.



UNIT OF USE

Unit of Use allows the operator to record and track manually counted pills; Eyecon counted pills; and, full bottle counts within the Report Database.



AUDIT REPORTS

Audit reports allow staff to analyze various aspects of prescriptions filled by the Eyecon. The Eyecon allows you to search:

- C-II prescriptions;
- Specific Drug Numbers;
- Specific uses;
- Partial-fills;
- and/or types and number of prescriptions filled per day.





The Eyecon is simple & easy to use.

eyecon®

The Eyecon is simple and easy to use which means your staff will need minimal training to have maximum benefits. This free's up pharmacy time and increases confidence and morale.



1

Scan Label - Scan barcode containing Drug Number, quantity, and Rx number. Or, with an interface, scan the existing barcode already on your vial label.



2

Scan Bottle - Scan barcode on stock bottle to validate that you've pulled the correct medication or be alerted to a generic substitution.



3

Verify - The Eyecon will proceed to count mode if the Rx label matches the stock bottle. The Eyecon will display an error if a mismatch is detected.



4

Count Pills - Pour pills onto tray until display turns green, indicating a match between required number of pills and number of pills on the tray.



5

Dispense - Press the Finish button to record the transaction. Then lift platter to slide pills into funnel.



6

Fill Vial - Use vial to lift funnel gate and allow pills to flow into vial.

SIMPLE & EASY

I have three pharmacies and an Eyecon in each location. Everyone loves it and would not want to work without it. Hands down this is the best investment in technology that I have ever made. We affectionately call it "Wally".

Eyecon has helped tremendously in our store. We fill about 180 prescriptions per day so a larger robot would not be appropriate for our business. I would estimate the Eyecon frees me up about a half of an hour a day extra to focus on my business with patients.

Been using an Eyecon for about 2 years. Best money spent, as it saves time, money and reduced errors. Never a complaint about miscounts now, if there is, we just show them the

What price do you put on your time?

Eyecon® is the preferred choice of pharmacies looking to:

- Ease pharmacy workload.
- Improve pharmacy revenue.
- Focus on patient safety.
- Enhance customer satisfaction.
- Reduce workplace stress.
- Improve work life balance.



Scan here to see a growing list of endorsements from pharmacists across the country.



How does the Eyecon® help?

UNPARALLELED SPEED

Speeding up the Rx filling process & reducing patient wait-time by up to 80%.

99.99% ACCURACY

Pill count accuracy 24/7, 365 days a year.

PEACE OF MIND

- ✓ Rx
- ✓ Drug & Strength
- ✓ Pill Count

OPTIMIZATION

Increase the accuracy and speed of physical inventories by over 70%.

EASY TO USE

Intuitive and ready to go straight out of the box.

AUDIT ASSISTANCE


Easy to access, digital records for audits and patient claims.

ORGANIZATION

Improve the accuracy of keeping track of partial fills.

RECORD KEEPING

Reduce legal liability through exact record keeping.

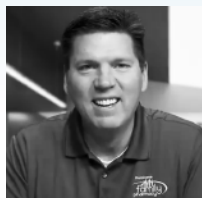

The first piece of equipment I purchase for my pharmacies is an Eyecon. I have 10 Eyecons in my stores and I cannot imagine what life would be like without



eyecon®



Comparison Chart



EYECON FUNCTIONS	EYECON 9420	EYECON 9430
Telepharmacy Enabled	-	✓
Color Drug Count Image	-	✓
Color Vial Image	-	✓
Color Labeled Vial Image	-	✓
Funnel NOT Empty Warning	-	✓
2D Barcode Scan	✓	✓
Fractional Qtys	✓	✓
Remote Auditor Software	✓	✓
Parse GS-1 Barcodes	✓	✓
Mckesson Software Interface	✓	✓
Cardinal CIM Interface	✓	✓
Imprivata Dual Authentication for User Login	✓	✓
Trained Drug Import/Export	✓	✓
Audit Trail	✓	✓
Enhanced Tray Photo Annotations	✓	✓



Eyecon by Avery Weigh-Tronix

1000 Armstrong Drive
Fairmont, Minnesota 56031 USA
Phone (866) 260-6540
EyeconOrders@awtx-itw.com

eyecon[®]